

AED/Crash Cart Daily Check Log Year 20 _____

Enter initials and crash cart lock ID if ALL of the following are true. If not OK, notify immediate supervisor.

- **Defibrillator Unit** - Clean, no spills, clear of objects on top, casing intact
- **Cables/Connectors** – Inspect for cracks, broken wire, or damage. Connectors are engaged securely and are not damaged.
- **Supplies** – Two sets of pads in sealed packages within expiration date, hand towel, scissors, razor, alcohol wipes, spare battery,
- **Power Supply** – Verify non-rechargeable battery is inserted and within expiration date. Verify system ready indicator is blinking green.
- **Crash Cart** – Lock is intact and ID matches previous day's ID. If not, inventory entire cart & forward missing items list to immediate supervisor.
- **Oxygen tank** – Oxygen gauge needle is within "Full" area. • **Suction Machine** (noted as **S.M.**) – monthly, turns on when plugged into outlet.

Day	Jan	Feb	Mar	Apr	May	Jun	Jul	Aug	Sep	Oct	Nov	Dec
1												
2												
3												
4												
5												
6												
7												
8												
9												
10												
11												
12												
13												
14												
15												
16												
17												
18												
19												
20												
21												
22												
23												
24												
25												
26												
27												
28												
29												
30												
31												
S. M. ✓												